**MICRO
SMART**

SUSTAINABLE PROGRESS

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ADAPTABLE PERFORMANCE

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When working with one single chair, Micro Smart is designed to have three different preset pressure settings: 70-100-210 millibar. Such values have been set according to some experts suggestions. Different settings however can be programmed by the operator anytime.

By pressing the desired preset button on the touchpad control, the head can be easily set according to the type of procedure to be performed. By means of the WI-FI connection the system can be controlled remotely. Micro Smart is ideal for two dental units working simultaneously as well. In case of simultaneous use the three preset pressure values cannot be used. The flow rate of the suction unit is sufficient for 1 or 2 chairs* working simultaneously. During operation the head remains constant whether only one or both operators are working, and any minor variation shall not be noticed.

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*according to the local professional common habits; i.e. in Italy and Spain the machine is suggested for two chairs, whereas in France and Great Britain for only one. Please contact your local dealer for advices.

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Micro Smart is an original and **innovative** machine with **6 patents**. Micro Smart is incredibly **reliable** in the event of a functional problem, the **active self-protection** feature electronically adjusts the unit's operation, resolving the problem without damaging the machine.

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70 MBAR

Is recommended for the use of the saliva ejector to maintain a low level of fluids at the base of the mouth, without damaging delicate tissues.

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100 MBAR

Is recommended for the use of the HVE spray interceptor which aspirates the stream of water and air from the syringe and rotating instruments, such as the sprays from scalers. Low pressure reduces discomfort and noise caused by the aspirated air to a minimum.

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210 MBAR

Recommended for the use of the surgical tip, which keeps the operating area clear and provides good visibility.

WI-FI CONNECTION

Your unit can be connected wireless to the computer in the clinic. The dentist can check and modify the functioning of the aspirator at any time without moving from his practice. Modern technologies are indispensable to obtain new work performances such as: planning and control of the prevalence, constant flow and prevalence during operations, connection with the computer in the surgery and with the after-sale service. We especially recommend the wi-fi connection: it is at no extra charge and can assist in preventing the unit from suffering due to the user unaware of malfunctions and continuing to use the unit.

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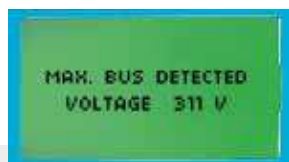
ACTIVE SELF-PROTECTION

Thanks to modern electronic and informatic technology, in case of difficult weather, feeding and use conditions, variable speed aspirators will go into active self-protection and they will not stop whereas the fixed speed aspirators will.

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Thanks to the inverter control and computerised program Micro Smart is able to handle a range of unexpected situations.

1. In difficult operational conditions, for example high temperatures, Micro Smart continues to operate without any damage to the unit. It does this through automatically decreasing the vacuum level until the temperature returns to normal.
2. In the case of excessive fluids from the surgeries entering the Micro Smart, an auto-protection system operates, momentarily opening a re-circulation valve, thus stopping the unit from being flooded. This also causes a momentary decrease in the speed of the suction motor (the system detects an increase in amperage to the centrifugal separator and reduces the speed of the suction motor), with the surgeries only noticing a minimal fluctuation if any at all, as the system goes back to its previous speed once the fluids are processed.
3. In cases of sudden voltage fluctuations and/or current overload within the system's parameters, the system continues to operate, thanks to Micro Smart's computerised control.
4. Possible operating anomalies are shown on the display during operation and will be recorded. This 'self-diagnosis' greatly helps clinic staff and service technicians to identify and rectify an issue.

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A DISPLAY THAT COMMUNICATES

Through simple and easy operations on the keypad you can monitor the operating status of the suction system. The unit's software monitors a range of technical parameters, allowing precise analysis of any possible anomalies. The unit also has a memory function, which records a log of any errors.

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Sound-proof box for indoor installation and **box for outdoor installation** with double insulating roof and aspirator operation temperature control are also available.

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RESPECT FOR OUR ENVIRONMENT THROUGH
EFFICIENT USE OF ENERGY. SUSTAINABLE PROGRESS.

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YOUR SUCTION SYSTEM CAN ALSO PLAY A PART IN CREATING A BETTER WORLD.

For some time researchers have warned us that our present consumption of materials and energy is causing an excessive exploitation of our natural resources. The less of these precious resources we are able to use, the greater is the chance that our ecosystem can regain its original capacity to provide them on a longterm and sustainable basis. On the other hand, the more of these resources we consume, the more our ecosystem's ability to supply them is diminished.

Along with many of our choices as a company, our research and development has been guided by the belief that through consistent effort by us all our resources can be conserved for the benefit of future generations.

MICRO SMART IS A SUCTION SYSTEM DESIGNED AROUND THE PRINCIPLES OF ECO-SUSTAINABILITY:

- reduction in materials used to produce the unit;
- the use of VSD (Variable Speed Drive) technology results in longer motor life, reduction in electrical consumption and operating costs;
- unique flexibility and new performance options to the clinic that were previously unavailable.

Micro Smart's suction motor weighs only 7 kg, however thanks to VSD technology it outperforms suction motors of double its weight and dimensions. The VSD facilitates optimal utilisation of the motor and reduces operating costs. The energy saving is most evident when the suction is underutilised; when the demand on the suction decreases the VSD reduces the power frequency to reduce speed. A reduction in motor speed results in a reduced energy absorption by the motor, providing a saving in electrical consumption.

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WE HAVE BEEN SPECIALISING WITH AIR TECHNOLOGY
FOR 50 YEARS:
SPECIALIZATION HAS GIVEN EXCELLENT RESULTS.

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HOW IS IT WE LEAD IN OUR FIELD, WHEN WE COST LESS THAN THE ALTERNATIVES? THIS IS HOW:

Constant research: this enables us to apply the latest technology to all of our products and solutions.

We enhance performance: electronic and information technology enable us to enhance the performance and reliability of our products.

We reduce costs: less maintenance and lower energy costs mean that we are always the most economical on a cost-benefit analysis.

We reduce environmental impact: we save 50% on raw materials, so that you can save between 30% and 50% on electrical consumption.