DUST ASPIRATOR

- Instruction Manual
- Manuale di Istruzioni
- Gebrauchs Anweisungen
- Mode d'emploi
- Manual de manejo

CAUTION:

- **INSTALL THIS DEVICE IN A PROPER LOCATION**
- **TO BE USE BY AUTHORIZED AND INSTRUCTED PERSONNEL ONLY**

Index

- Declaration of conformity
- Visual Instructions
- Exploded draw
- Spare parts list
- Electrical diagram

Instruction Manual

1. Safety directives
2. Unpack
3. Installation
4. Maintenance
5. Guide to common questions
6. Specifications
7. Dimensions

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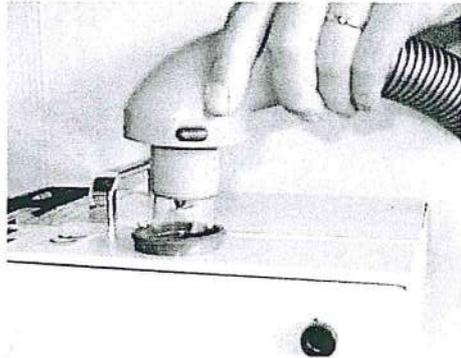
ESPAÑOL

Visual Instructions


Please read instruction booklet carefully before installing and use of this device

1


Insert air hose connector on plastic ring on top

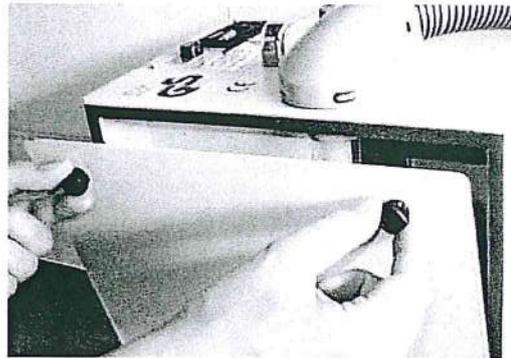2


Remove lateral panel to install the filter

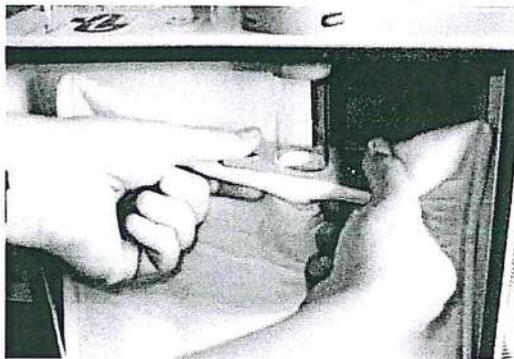3


Place original filter (Airbag) on transparent hose

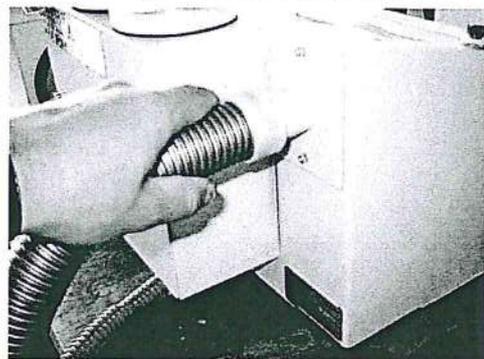4


Connect air hose to selected device or bench

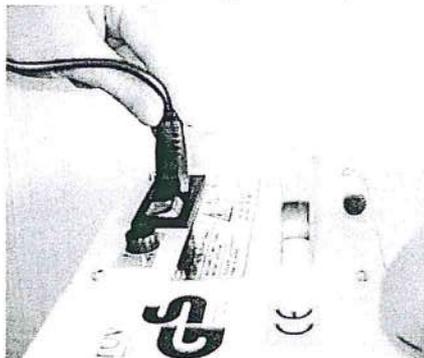5


Insert plug respecting voltage on back plate

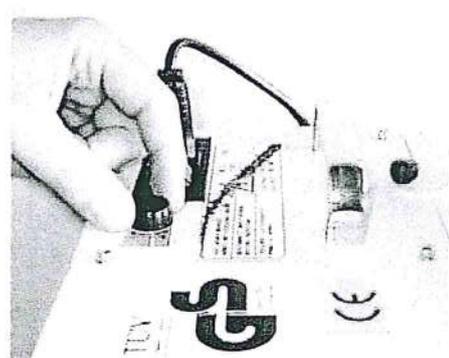6


Regulate power (minimum for sandblasters)

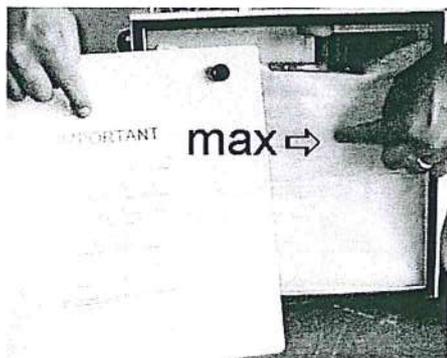7

CAUTION: replace Airbag filter before it is full

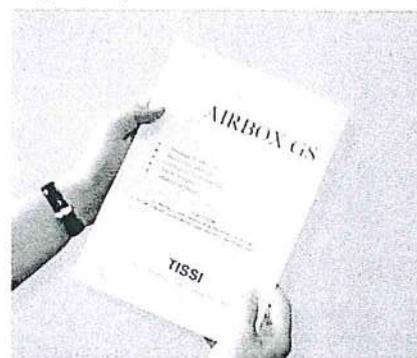8

Consult instruction booklet before operate

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IN BASE ALLE DIRETTIVE:
as for directives:

89/392/CEE
89/336/CEE
73/23/CEE
89/68/CEE

E SUCCESSIVE MODIFICHE
And subsequential modifies

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
MATRICOLA: _____

Matriculation nr.

E' CONFORME ALLE NORME:
is made in conformity with normatives:

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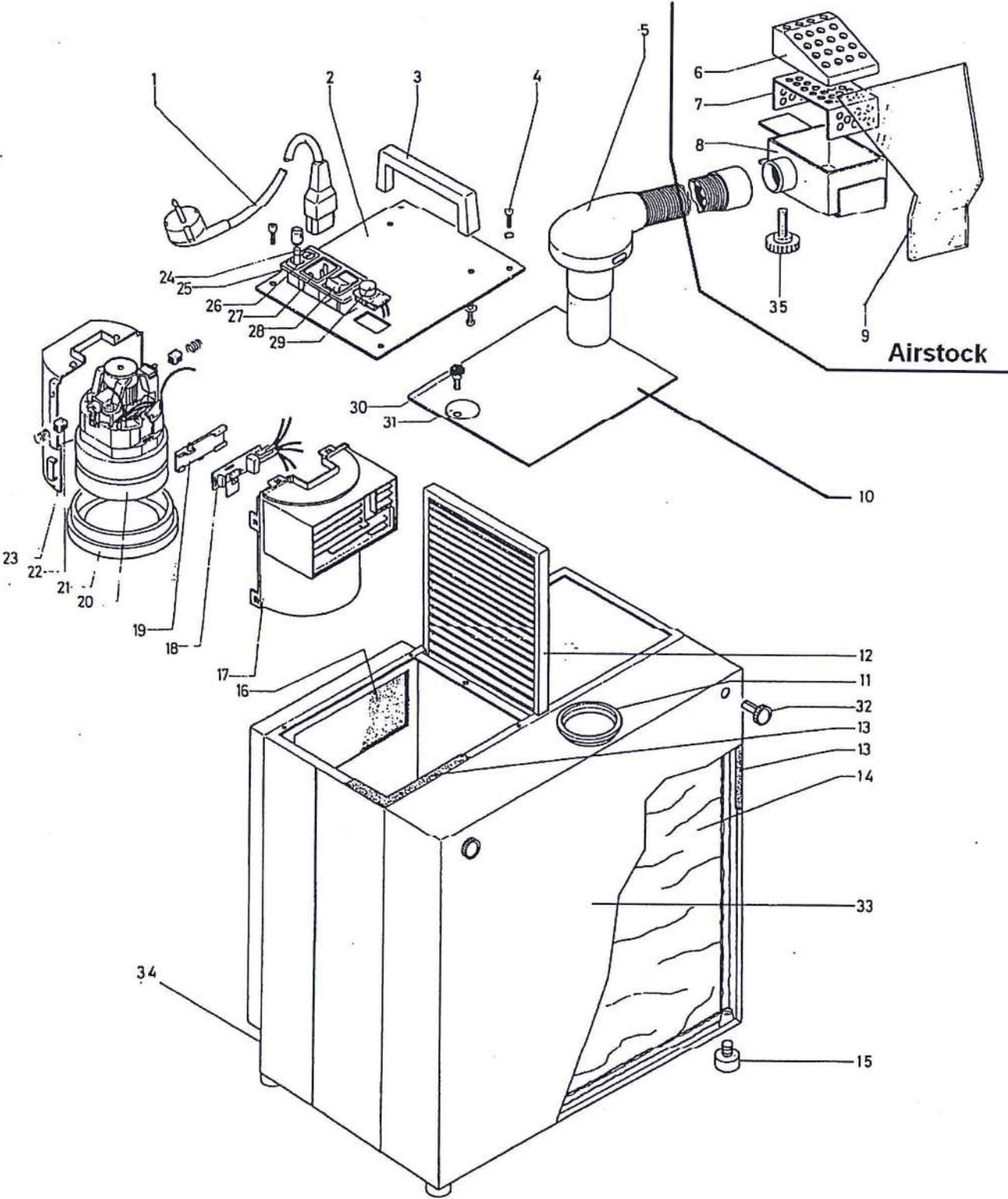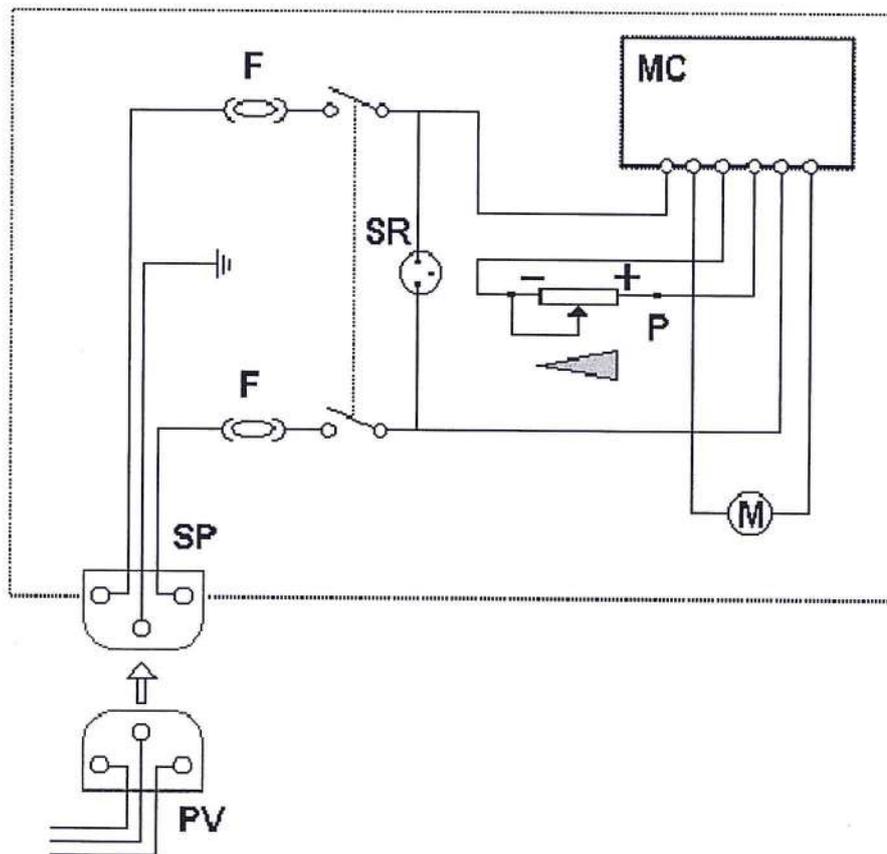
CONGRATULATIONS FOR YOUR CHOICE!!

ENGLISH

Choosing this apparatus you have purchased a particularly reliable unit that will facilitate your work.

This unit complies with the directives 89/392/CE, therefore CE mark is admitted:

IMPORTANT!

Before installing the apparatus read carefully this manual and keep it for future consultations.

WARNING

Do follow carefully all the indications brought back in this manual. During all the maintenance operations disconnect the power cord from the electrical outlet, and/or disconnect the compressed air and/or the water attack and/or other attacks of external parts to the apparatus. Do not introduce stranger objects inside the unit during the maintenance operations. Use only the accessories in equipment for the filling operations and/or maintenance. Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not contemplated in this manual, can involve risks. For operations of this type, contact the technical service. Before turning on the unit and connecting the power cord to the electrical outlet, make sure that the voltage corresponds to the one brought back on the fabrication nameplate placed on the back part of the apparatus.

CAUTION:

- **Do not make any maintenance operation when the power cord is connected: FULGURATION DANGER!!!**
- **Use only genuine spare parts. Placing not genuine spare parts can void your warranty, moreover can make the apparatus not in compliance with the norms, and dangerous when in use.**

For whichever inherent information concerning installation, maintenance and use of the unit, contact the customers' service

UNPACK

Verify integrity of the pack at the moment of the reception of the apparatus. Verify that in the packaging all the parts and the supplied accessories of series are contained together with the apparatus: power cord, suction hose and connection, pack of 4 filters AIRBAG (rif. 14 - conf. 4 pz.), user's documentation and warranty.

INSTALLATION

Take the suction unit out of the packaging and place it where can be easily inspected for the replacement of the recovery filter of powders AIRBAG (rif. 14) and for the periodic substitution of the filter for motor protection (rif. 12). ATTENTION: do not place the apparatus inside a piece of furniture without circulation of air.

Open the lateral panel (rif. 33) rotating the two screws (rif. 32).

Insert the suction hose making a light pressure (rif. 5) in the appropriate metal ring (rif. 11).

Make sure of the proper placement of the recovery filter of powders (rif. 14) in the appropriate plexiglass connection of the suction hose (rif. 5), then close the lateral panel (rif. 33).

Connect the power cord (rif. 1) in the plug of the unit (rif. 27) and the other head of the cord to a power outlet (absorption max 1000 W). ATTENTION: make sure that the voltage corresponds to the one brought back on the fabrication nameplate placed on the apparatus.

Connect the other side of the suction hose (rif. 5) where you need to have the aspiration: AIRSTOCK - TISSI (rif. 6/7/8/9/35 - cod. 110), benches, sandblasters, dry trimmers (for the use of the suction unit with a dry trimmer it is recommended to connect it to a decantation box for dry powders type TRIMBOX - TISSI).

After these operations, the suction unit is now ready to work: turn on the apparatus by pressing the general switch (rif. 28) and regulate the aspiration power acting on the power regulator (rif. the 29) min 400 W - max 1000 W.

ATTENTION: the AIRBAG filter (rif. 14) has a very important function for the perfect operating of the apparatus and for your health. Check it periodically and replace it before it is completely full. Do not use the same filter more than one time, and do not discharge it. Non-observance will void your warranty. Once exhausted the filter close it with a proper adhesive tape and waste appropriately in compliance with your country rules.

IMPORTANT INFORMATION

- The Manufacturing Company declines any responsibility for any apparatus installed, utilized, modified, repaired and/or altered by non competent and unauthorized personnel.
- The apparatus has been designed and built only for the uses contemplated, consequently the Manufacturing Company refuses any responsibility for damages to persons, animals, things, and the machine itself caused by improper use of this apparatus.

The improper use of this machine is dangerous!!

- Subject the apparatus to a complete annual revision by the Manufacturer.
- To clarify any doubt and/or technical explanation please contact our Technical Department.

MAINTENANCE

Substitution of the used filter of powders:

Open the lateral panel (rif. 33) of the suction unit removing the two screws (rif. 32), control the bag of used powders (AIRBAG - pack 4 pcs. - rif. 14) every 4/5 days and replace it before completely full (max to approximately 2/3 of its capacity). Place a new filter bag making a light pressure upwards until it is perfectly fitted in the plexiglass connection (rif. 5). Do not re-use and do not empty out the used filters. When the filter is exhausted close it with a proper adhesive tape and waste appropriately in compliance with your country rules.

Substitution of the motor protection filter:

Remove the metal central panel screw (rif. 30), rotating the metal holder disk (rif. 31); pull and extract upwards the filter. Insert a new filter (rif. 12 - cod. 3106012) and close the all making operations to the contrary. Check every two weeks the motor protection filter and clean it by blowing compressed air in the same way of passage of the sucked air. The filter must be replaced with a new one at least every 4 months. This preserves the motor and the perfect operating of the apparatus.

GUIDE TO COMMON QUESTIONS**The apparatus is not turning on:**

1. Control that the power cord plug (rif. 1) is properly connected to the electrical outlet.
2. Check that fuses (rif. 24) are not damaged.

The suction power is low:

1. Replace the recovery dust filter (rif. 14), and/or replace the motor protection filter (rif. 12).
2. Check that the motor is correctly placed in the center of its site (rif. 17a23).

Carried out these controls, in case the apparatus still presents disadvantages, contact the customers' technical service.

SPECIFICATIONS

Fuses: 2 fuses 5x20 mm. 6,3 Ampere

Power max 1000 Watt.

Operating voltage V/Hz 230/50-60 (see nameplate on the apparatus)

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