01

SIDÈREA

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USER MANUAL AND INSTALLATION

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INDEX

1	SAFETY REQUIREMENTS.....	17
1.1	Symbols used.....	17
1.1.1	Symbols used in the manual.....	17
1.1.2	Symbols present on product labelling.....	17
1.1.3	Product symbols.....	17
1.1.4	Symbols present on the packaging.....	18
1.2	Designed use.....	18
1.3	General regulations and main warnings.....	19
1.4	Preservations and use; environmental prescriptions.....	20
	GENERAL CHARACTERISTICS.....	21
2.1	Product description.....	21
2.2	Description of the parts.....	21
3	DEVICE INSTALLATION.....	22
3.1	Dimensions.....	22
3.2	Positioning.....	23
3.3	Affixing to the ceiling.....	23
3.4	Installation procedure.....	23
4	INSTRUCTIONS FOR USE.....	24
4.1	Method of use.....	24
4.2	Remote control commands.....	25
4.3	Substitution of remote control battery.....	26
4.4	APP for mobile phones and tablets.....	26
4.5	Sidèrea synchronization with dental lamp.....	26
4.6	“Pairing” procedure.....	27
5	MAINTENANCE.....	28
5.1	Ordinary maintenance.....	28
6	CLEANING.....	28
7	PROBLEMS GUIDE.....	29
8	TECHNICAL SPECIFICATIONS.....	30

1. SAFETY REQUIREMENTS

Dear Client,

FARO hopes you enjoy your work with the new high quality light. For safe work and to take full advantage of the performance of the product, carefully read this manual before using the device. In particular, follow all the warnings and the notes given.

1.1 SYMBOLS USED

1.1.1 Symbols used in the manual

	WARNING
Paragraphs highlighted with this symbol contain instructions which must be carried followed out with care to avoid damage the device, the operator or the patient.	
	WARNING
These instructions warn of the need to take particular care to avoid situations which might damage the device.	
	PROHIBITION
This icon places in evidence what must not be done, in order to avoid damage to the device.	
	NOTE
With this icon, information is given which allows for using the device in a more effective manner.	

1.1.2 Symbols present on product labelling

The data label is affixed to the upper part and contains the following data:

Serial Number (SN): year (YY) / device family (AL) for light environment, plus progressive number (NNNNNN) e.g.: SN17AL000001


There are present the following harmonized symbols:

	Manufacturer's symbol
	The instructions for use include safety warnings
	RAEE equipment in compliance with the Directive 1012/19/EC. Dispose of the product in compliance with this directive.
	Serial Number
	Use the device at a temperature between 10°C and 32°C
	Use the device at pressure between 80 kPa and 106 kPa
	Use the device at relative humidity between 30% RH and 75%RH

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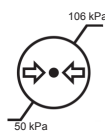
ATMOSPHERIC PRESSURE

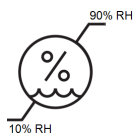RELATIVE HUMIDITY

**STORAGE TEMPERATURE
TRANSPORT CONDITIONS**

RECYCLE-ABLE CARTON

Kg 300 MAX

MAX STACKABLE WEIGHT

UPPER

FRAGILE

DO NOT WET

DO NOT ROLL

DO NOT USE HOOKS

1.2 *DESIGNED USE*

The lamp is designed to illuminate work environments including medical surgeries.
The device must be installed on the ceiling by qualified personnel.

1.3 GENERAL REGULATIONS AND MAIN WARNINGS

- The lamp packaging is ideal to protect it from penetration from external agents.

	The ceiling must be able to support 40 kg for each fixing point.
---	---

	Warning against electrical or fire danger
	Do not use the lamp in the event of damage to its components Installation of the device must be carried out by qualified personnel only. The device must be installed with a network omni-polar separation device, conforming with Standard IEC 61058-1 and equipped with an earthing cable. Verify that the power voltage indicated on the data label corresponds with that of the network. Do not carry out any maintenance on the lamp when the power is on. In other words, disconnect the power cable before intervening.
	Warning of falling danger from suspended mass
	Comply scrupulously with the established maximum suspended weights.
	Photobiological risk warning
	The lamp is classified as Exempt from photobiological risk in accordance with EN 62471. Nonetheless, it is not excluded that particularly photo-sensitive patients, or those who take photosensitive medicines, might suffer erythemas or allergic reactions to the light. In this case, suspend treatment and use low levels of illumination.
	Warning of the danger of explosion
	The device is not suited to being installed with the presence of inflammable gas or excess of oxygen.
	Warning of the danger of incorrect maintenance
	Do not carry out maintenance operations or substitution of parts which differ from those contained in the manual. Any intervention not indicated in the same might compromise the safety aspect required of the device. Only carry out maintenance operations contained in the manual; for any other case, contact technical support.
	Warning of the danger of damaging
	In order to avoid damage to the product, do not push or force the light diffusion panel.

	Disposal warning
	The product is covered by Directive RAEE 2012/19/UE For scrapping or disposal of materials, comply with the legislation applicable in your country, possibly through the use of recognized and authorized companies. At the end of the life cycle, divide the materials according to type (ferrous, rubber, plastic). Do not leave small components of the equipment unguarded or in the reach of exposed persons (children), as they may be a source of danger.

The FARO company does not accept any modification to the product which has not been expressly authorised in writing, on penalty of forfeiture of the safety standards and the guarantee.

1.4 PRESERVATION AND USE: ENVIRONMENTAL PRESCRIPTIONS

The device in its original packaging may be transported or held in storage for a period of 15 weeks as long as the following environmental conditions are complied with.

- **Environmental temperature between -20° and + 70°C**
- **Relative humidity between 10% RH and 90% RH**
- **Atmospheric pressure between 50 kPa and 106 kPa**

The equipment must be used under the following environmental conditions:

- **Temperature of between 10° and 32°C**
- **Max. altitude of: 2000m**
- **Relative humidity between 30% RH and 75% RH**

2. GENERAL CHARACTERISTICS

2.1 PRODUCT DESCRIPTION

The device is designed for illuminating the work environment

The light source is made up of LEDs whose light diffuses through specific diffusers.

The diffusers allow for a regular and uniform light spot at all levels of illumination, and to distribute the light uniformly throughout the environment without creating shadows or obstructions by the operator.

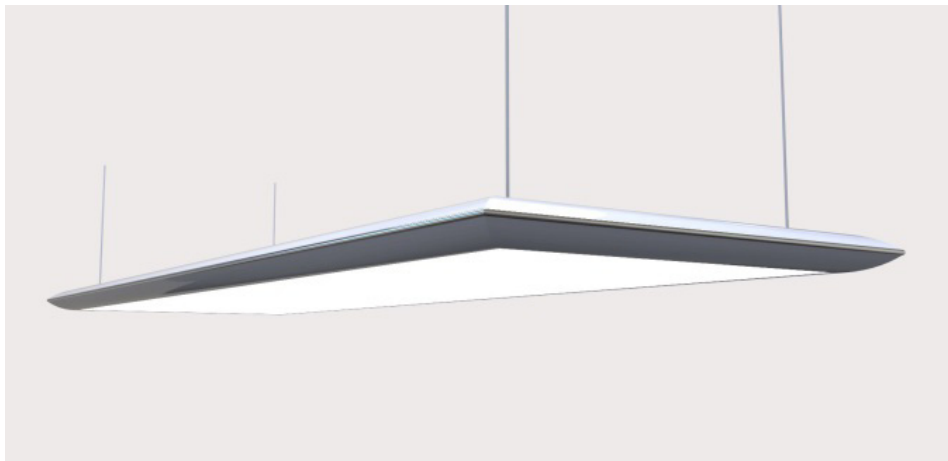
Adjustment of light intensity must be carried out with the remote control and may be coordinated

with the Alya dental light (where foreseen).

For electrical connections, keep to the instructions contained in the installation paragraph.

2.2 PARTS DESCRIPTION


Sidèrea Lamp


Remote control

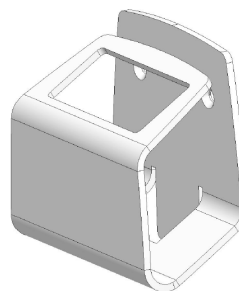Remote control support

Fixing elements

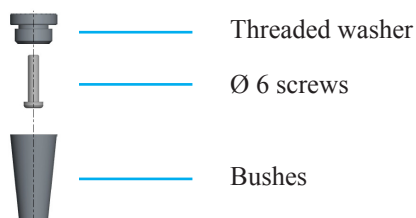3. DEVICE INSTALLATION

	Warning of electrical dangers, for the lamp and the head:
	The device must be installed by specialist technicians.
	During the installation, the general power source to which the lamp must be connected must be disconnected.
	For the connection use a terminal block conform to the EN 60998-2-1 or EN60998-2-2 E.G: Cembre Z205-1 Zetamini serie Check the label data before installation.
	Notes for installation
	The lamp's operation or safety does not depend on the polarity or the power source. For which, inversion of the power cables poses no risk of malfunctioning.
	Notes for installation
	Suspend/lift the lamp using the outer casing. Do not apply force on the panel diffuser (white).

3.1 DIMENSIONS

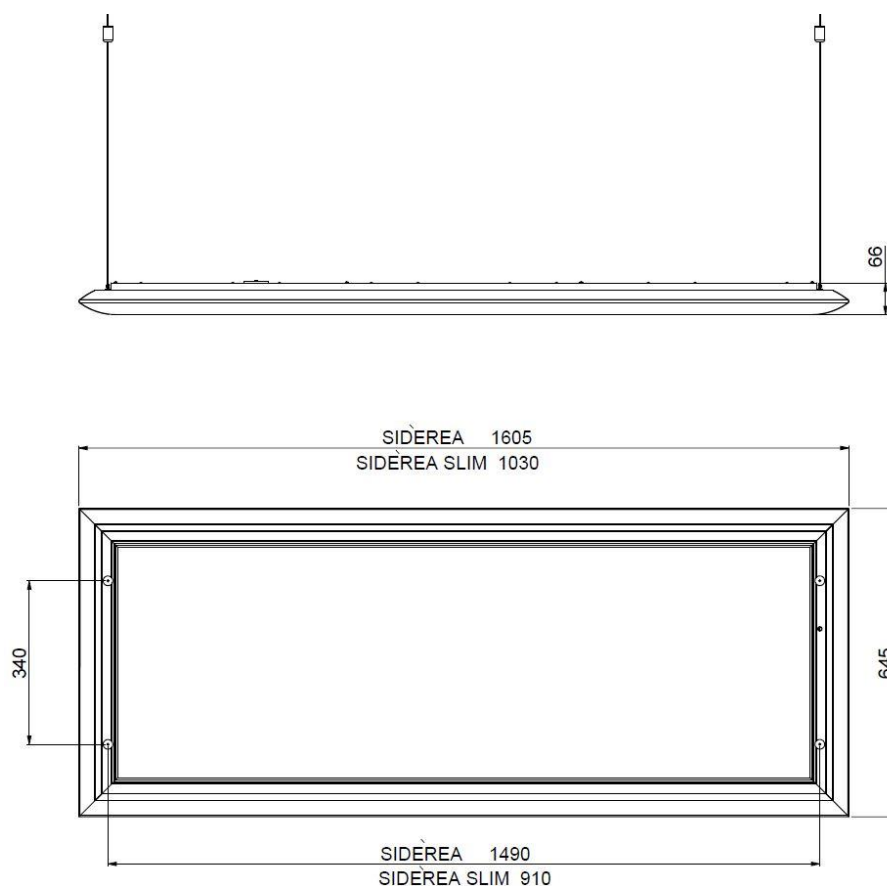
Fig. 1

3.2 POSITIONING

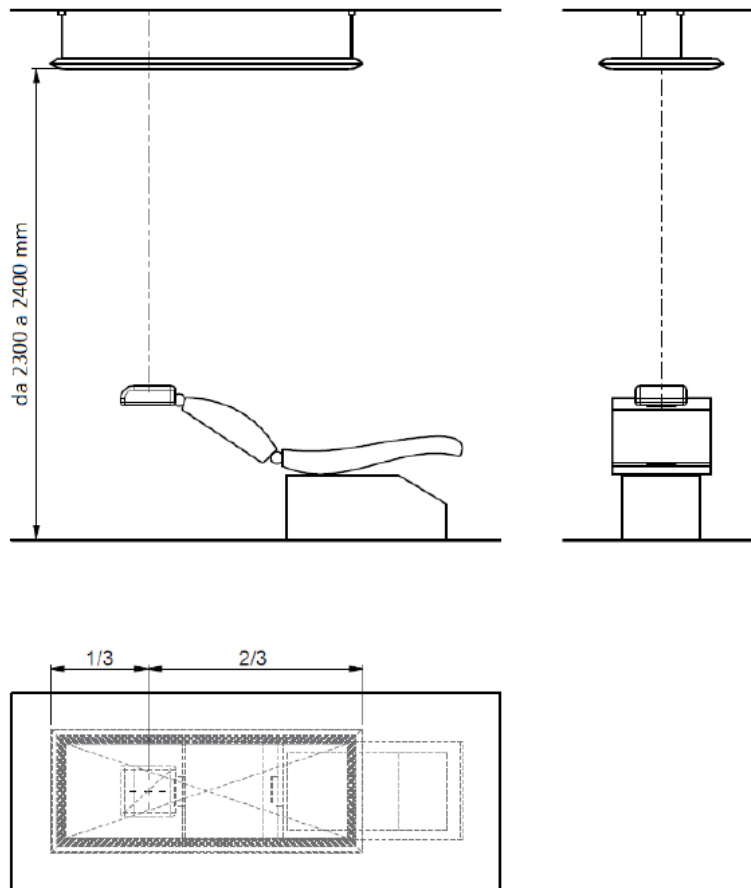
Fig. 2

3.3 AFFIXING TO THE CEILING

The Sidèrea lamp is complete with elements for affixing to the ceiling.

Included:

- Sidèrea Lamp
- 4 plugs with Ø 6 mm screws
- Threaded washers plus bush
- 4 steel cables for hooking the environment lamp to the ceiling

3.4 INSTALLATION PROCEDURE

- Drill the ceiling at the required position with a suitable bit (for heights, see 3.1-fig.1) and insert the plugs.

FIG. 3 Ceiling

- 1) Position the threaded washers at the holes drilled into the ceiling
- 2) Block the washers with Ø 6 screws
- 3) Thread the steel cable into the bushes and screw them to the threaded washers

FIG. 4 Sidèrea Lamp

- 1) Partly unscrew the knob
- 2) Press the knob
- 3) Insert the steel cable
- Hook the lamp up using the system of steel cables
- Fix the height of the lamp at a height of between 2200/2400 mm.
- To adjust the height, use the 4 knobs situated on the upper part of the environment lamp
- Check that it is level

	Note for installation
	Verify the grip of the plugs in the ceiling by applying an axial load of at least 15 kg. Failure to comply with these prescriptions may result in a detachment of the product with the risk of falling onto persons.

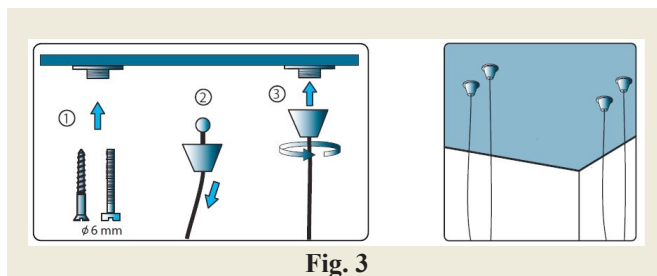
Fig. 3

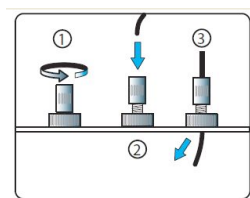
Fig. 4

4. INSTRUCTION FOR USE

4.1 METHOD OF USE

The lamp is lit and is adjusted by use of the issued remote control provided or using the free application (App) downloadable from Play Store (Android devices) or Apple Store (Apple devices).

Switching on and turning off may also be occasioned by use of the omni-polar switch present in the surgery.

4.2 RADIO CONTROL COMMANDS

Buttons	Functions
	On/Off: switching on/turning off the lamp. On switching on, the last scenario selected will be recalled
	Programming button; allows access to “overwrite” and “Reset” functions.
	Scenario 1: 1st level of illumination, maximum + upper LEDs (min.)
	Scenario 2: 2nd level of illumination, maximum + upper LEDs
	Scenario 3: 3rd level of illumination, maximum + upper LEDs
	Scenario 4: 4th level of illumination, maximum + upper LEDs
	Scenario 5: 5th level of illumination, maximum + upper LEDs (max.)
	Adjustment to lower LEDs; variation in intensity of 0% to 100%. An increase/reduction in intensity depends on the time for which the button is pressed by the user.
	Company Logo, no function

	Overwriting scenario: Modification of the level of illumination over an existing scenario. <i>Procedure:</i> 1. Set the desired level of illumination. 2. Press the two indicated buttons at the same time for 3 secs.; that of the scenario to be modified and the programming button, the green LED flashes 3. Release the buttons as soon as the blue light is steady. <i>Note: The Modification is possible on all 5 scenarios</i>
	Restore initial settings; eliminates all modifications carried out. <i>Procedure:</i> 1. Prolonged pressing of the two buttons for some 5 secs., the green light flashes. 2. Release the buttons as soon as the green light is steady.
	Modify the coding of the remote control; create a new code for communicating between the remote control and the lamp. 1. This function must be only be used in the event that it is necessary to pair a new remote control in the place of the one in use. <i>Procedure:</i> 1. Remove power to the lamp. 2. Restore power and within 60 seconds, proceed with points 3 and 4 3. Prolonged pressing of the two buttons for some 5 secs., the green light flashes. 4. Release the buttons as soon as the green light is steady. 5. When the green Sidèrea LED flashes rapidly, press the programming button

4.3 ***SUBSTITUTION OF THE REMOTE-CONTROL BATTERY***

The battery is accessed through a hatch at the back of the remote control. Exchange it for one with the same characteristics, observing the indicated polarity:

MOD: LR-23A/12V

4.4 ***APP FOR MOBILE PHONES AND TABLETS***

The Sidèrea App allows users to switch the lamp on and off, adjust the intensity of the light and set 5 pre-configured modes.

Note: Unlike the remote control, the App does not allow to overwrite or reset the settings.

Downloading the App

The (free) App can be downloaded from both Play Store (for Android devices) and Apple Store (for Apple devices).

Connecting to the lamp

Once you have downloaded the App on your device, follow the instructions to install it. Once the App is installed, when you open it you will see the screen below.

To connect it to the lamp:

1. press the bluetooth button in the top right corner and wait for the Siderèa lamp to be identified.
2. Once the lamp has been identified, select the device and request the connection.
3. Enter the required PIN, which is the last 4 digits of the serial number the Siderèa lamp shows during the connection stage.

Note: if the lamp is not identified, repeat step 1.

Use.

- ✓ On/Off: press and release the button in the centre. Note that in this case the lamp remains on stand-by, to switch off the lamp completely, use the omnipolar switch of the electrical system.
- ✓ Modes: press the icon corresponding to the desired mode, the selection cursor will move to the selected position and the bar below that indicates the illumination will move to the relevant working position.
- ✓ Info: press the icon in the top left corner. The screen will show the serial number and product code you will need to provide if you require post-sales support

4.5 ***SYNCHRONIZATION WITH THE DENTAL LAMP***

Where foreseen, it is possible to connect the environment light to the dental light in wireless mode, in order to turn the both into a synchronised lighting system named “**Synchro**”.

The “**Synchro**” mode has been purposefully designed to improve the medical dental/orthodontal comfort in order to reduce the light blindness which occurs when passing from observing under a strongly lit surface (e.g. oral cavity with the dental light) to a poorly lit surface (e.g. dental delivery system).

The “**Synchro**” mode, button situated on the dental lamp, allows for modifying the lighting value produced by the Sidèrea lamp automatically, on the basis of regulating the lighting produced.

In the “**Synchro**” mode, the dental light functions as the “**Master**”, given that it has control over that environment.

Note: Between the dental light and that of the environment, a small delay in synchronization may be experienced. This is due to the communications protocol. This effect is normal and does not signify a defect.

The “**Synchro**” function, in order to be enabled, requires a procedure called “**Pairing**” (which only needs to be carried out once) in order to create the link between the two lamps. Subsequently, the “**Synchro**” function may be enabled and/or disabled as required by the user using the button situated on the dental lamp.

4.6 “**PAIRING**” PROCEDURE

Note:

- The “Pairing” procedure is only necessary for the first connection, although it can be repeated in the event of substituting the dental lamp or the electronics of one of the two lamps in the system.
- If there should be more lamps in the surgery, ensure that the other lamps have been turned off or on more than 60 seconds previously.

To carry out the “Pairing”, proceed as follows:

1. **Apply power** to the Sidèrea environment light which is to be paired.
The environment light listens out/awaits for a maximum of 60 secs.
2. **Press the “Synchro” button on the dental light** for at least 3 secs., but not over 6 secs., otherwise the procedure will be aborted and the software continues as if the button had never been pressed. On receiving the “**Pairing**” request from the dental lamp, the environment lamp activates the green LED situated on the frame.
3. **Press the programming button  on the remote control** within 60 seconds of operation 2 (synchro). The green LED flashes twice and then goes out.

Note: If within the 60 seconds the button is not pressed,  the green LED turns off and the procedure must be repeated.

Having carried out the “**Pairing**”, synchronization between the 2 lamps is now active which, in any case, must be enabled by the dental lamp by pressing the “**Synchro**” for some 2 secs.

Note connected with synchronization:

- When the Sidèrea environment lamp is synchronized (or when it adjusts itself automatically) with the dental lamp, the green LED situated on the frame (Sidèrea) is lit and steady. When the LED is not lit, synchronization is deactivated.
- The remote control is always enabled and it is therefore possible to modify the lighting value. Nonetheless, if the environment light should be in the synchronized state (with the green LED lit), as soon as a new adjustment is made on the dental lamp, the lighting value will immediately be updated.
- If the dental light should be turned off, the environment lamp will remain lit at the current lighting value.
- If the dental lamp should be switched on, the environment lamp will turn on automatically.

5. MAINTENANCE

5.1 ORDINARY MAINTENANCE

There are no ordinary maintenance operations.

6. CLEANING

 	Warning against the dangers of degradation and corrosion The use of abrasive substances, acids, substances containing chlorides or Chlorine ions, trichloroethylene based detergents, petrol, white spirit or similar, is strictly forbidden on metal or plastic parts of the lamp. It is forbidden to spray any chemical substance directly onto the device.
 	Warning To clean the parts, do not use detergents-disinfectants containing: • AMMONIUM HYDROXIDE • SODIUM HYDROXIDE • METHYLENE CLORIDE • METHYL ALCOHOL • ACIDS OF ANY KIND
	Note for cleaning Do not press on the panel with force during cleaning operations.

7. PROBLEMS GUIDE

Effect	Cause	Action	Resp
The lamp does not light.	Mains power is missing.	Check the mains power supply.	User
		Check that the network voltage is correct.	User
The lamp does not adjust.	The remote-control battery is flat.	Change the battery.	User
	The distance between the environment lamp and the remote control is too great.	Return the remote control within range.	User
	The remote control used is that of another environment lamp.	Check with the other available remote controls.	User
	The remote control has been reset or has lost the internal code.	Carry out the pairing once more as described in the user instructions section.	User
The environment lamp automatically changes the lighting mode.	The “Synchro” function is active with the dental lamp.	Disable the “Synchro” function using the dental lamp control.	User
The light emitted from the environment lamp does not correspond with the remote-control scenarios.	The remote-control settings have been modified.	Carry out a reset of the scenarios using the remote control as described in the user instructions section.	User

For all other problems, contact the Faro Spa authorized technical services.

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GUARANTEE CERTIFICATE

FARO offers the final customer a **24 month** guarantee starting from the date of purchase.

Repairs under guarantee must be performed at FARO; expenses and transport risks are at the risk of the purchaser. **Repair under guarantee is considered valid only when:**

- **all sections of the certificate have been filled in and sent in advance to FARO by Fax (039.6010540).**

The guarantee covers faults due to the bad quality of the material or manufacturing defects; in the case of valid claims, the guarantee covers free repair or replacement. **Claims for damages and/or interest are excluded.** The guarantee is not considered valid, at the sole discretion of FARO, if the fault is due to tampering, damage, incorrect use, improper maintenance and normal wear and tear.

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FARO S.p.A.
FARO SpA si riserva il diritto di modificare, senza preavviso, le caratteristiche indicate nel presente manuale.
FARO SpA reserves the right to change the specifications of this equipment without notice.
FARO SpA se reserve le droit de modifier, sans préavis, les caractéristiques dans ce manuel.
FARO SpA behält sich recht vor, jederzeit stillschweigend technische oder bauliche Änderung vorzunehmen.
FARO SpA se reserva el derecho de modificar sin aviso previo la características incluidas en el presente manual de uso.