01

ZIVA

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USER MANUAL AND INSTALLATION

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: EN62471

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INDEX

1	SAFETY REQUIREMENTS	14
1.1	Symbols used	14
1.1.1	Symbols used in the manual	14
1.1.2	Symbols present in product label.....	14
1.1.3	Symbols on the packaging.....	15
1.2	Intended use.....	15
1.3	General rules and main warnings	16
1.4	Storage and use: environmental requirements	17
2	GENERAL CHARACTERISTICS	17
2.1	Description of the product.....	17
2.2	Description of the components.....	17
3	DEVICE INSTALLATION	18
3.1	Overall dimensions	18
3.2	Positioning	19
3.2.1	Suspended installation	19
3.3	Ceiling installation	19
3.4	Installation procedure	19
4	INSTRUCTION OF USE	20
4.1	How to use.....	20
4.2	How to use “CASAMBI” application.....	20
5	MAINTENANCE	23
5.1	Ordinary maintenance	23
6	CLEANING	23
7	TROUBLESHOOTING	24
8	TECHNICAL SPECIFICATIONS	24

1. SAFETY REQUIREMENTS

Dear Customer,

FARO hopes you enjoy working with the new high-quality light. In order to work safely and to get the best of the performance out of the product, read this manual carefully before using the device. In particular, follow all the warnings and the notes.

1.1 SYMBOLS USED

1.1.1 Symbols used in the manual

	WARNING
The paragraphs marked with this symbol contain instructions that must be followed carefully to avoid damage to the device, operator and patient.	
	CAUTION
These instructions warn that care must be taken to avoid situations that could damage the device.	
	PROHIBITION
This icon highlights what you should not do to prevent damage to the device.	
	NOTES
With this icon, information is provided that allows you to use the device in a more effectively.	

1.1.2 Symbols present on product label


The data label is attached o the top and includes the following data:

Serial Number (SN): year (YY) / lighting sector (AL) plus progressive number (NNNNNN)
e.g.: SN17AL000001


The following harmonized symbols are also present:

	Manufacturer's symbol in accordance with the Directive
	The instructions for use include safety warnings
	WEEE equipment according to directive 1012/19/EC. Dispose of the product according to this directive.
	Serial Number
	Use the device at a temperature between 10°C and 32°C
	Use the device at a pressure between 80 kPa and 106 kPa
	Use the device at a relative humidity between 30% RH and 75%RH

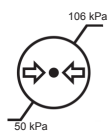1.1.3 Symbols on the packaging

ATMOSPHERIC PRESSURE

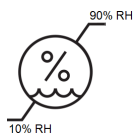RELATIVE HUMIDITY

**STORAGE TEMPERATURE
TRANSPORT CONDITIONS**

**RECYCLABLE
CARDBOARD**

Kg 300 MAX

MAX STACKING WEIGHT

**THIS SIDE
UP**

FRAGILE

**DO NOT GET
WET**

DO NOT ROLL

DO NOT USE HOOKS

1.2 INTENDED USE

The lamp is designed to illuminate work environments including medical surgeries.
The device must be installed on the ceiling by qualified personnel.

1.3 GENERAL RULES AND MAIN WARNINGS

- The light's packaging is suitable to adequately protect against external agents from entering.

	The ceiling must be able to support 40 kg at each fixing point.
	Warning against electrical or fire hazard Do not use the light if any of this components are damaged. The device must only be installed by qualified personnel. The appliance must be installed with an omnipolar type xut-off device in compliance with the IEC 61058-1 Standard, provided with a grounding cable. Check that the supply voltage indicated on the data plate corresponds to the mains voltage. Do not carry out any maintenance operations on the light when the power is on: disconnect the power cable from the mains before operating.
	Caution, risk of electric shock (source: IEC 60417-6042 (2011-11)) The light source contained in this luminaire shall only be replaced by the manufacturer or his service or a similar qualified person
	Warning of the hazard of falling of suspended masses Strictly observe the maximum sexpected loads. Photobiological risk warning The light is classified as a photobiological risk according to EN 62471. However, it cannot be excluded that patients who are particularly photosensitive or who have taken photosensitising medicinal products may have erythema or allergic reactions to the light. In this case, suspend the treatment and use very low levels of lighting.
	Warning of risk of explosion The device is not suitable for installtion in flammable or oxygen-rich environments.
	Warning of the hazard arising from incorrect maintenance Do not carry out any maintenance or replace parts otherthan those indicated in the manual. Any operation not indicated therein may compromise the device's safety. Only perform maintenance operations listed in the manual; in any other case, contact technical support.
	Warning of the hazard of damaging To avoid the risk of damaging to the product, do not push and/or force the light panel.
	Disposal notice The product is covered by WEEE Directive 2012/19/UE For the scrapping and disposal of materials, follow the regulations in force in your country, possibly using recognized and authorized specialist companies. At the end of the useful life, separate the materials according to their type (ferrous, rubber, plastic). Do not leave small parts of the appliance unattended or within the reach of exposed persons (children) because they are tpotential sources of danger.

The company FARO does not allow any modification to the product that is not expressly authorised in writing, under penalty of forfeiture of compliance with the safety and warranty regulations.

1.4 STORAGE AND USE: ENVIRONMENTAL REQUIREMENTS

The appliance in the original packaging can be transported or kept in storage for a period of 15 weeks if the following environmental conditions are met:

- **Environmental temperature from -20° to + 70°C**
- **Relative humidity between 10% RH and 90% RH**
- **Atmospheric pressure between 50 kPa and 106 kPa**

The appliance must be used under the following environmental conditions:

- **Temperature from 10° to 32°C**
- **Max altitude : 2000m**
- **Relative humidity from 30% RH to 75% RH**

2. GENERAL CHARACTERISTICS

2.1 DESCRIPTION OF THE PRODUCT

The device is designed for illuminating the work environment

The light source consists of LEDs whose light diffuses through specific diffusers.

The diffusers allow to obtain a regular and uniform light spot at any level of lighting and to distribute the light in the environment.

Adjustment of light intensity must be carried out with the “CASAMBI” application

For electrical connections, please follow the instructions contained in the installation paragraph.

2.2 DESCRIPTION OF THE COMPONENTS

ZIVA LAMP

Fig. 1

3. DEVICE INSTALLATION

	Warning of potential electrical, light and head hazard: The device must be installed by specialist technicians. The power supply must always be disconnected during installation operations. For the connection, use a terminal board in accordance with EN 60998-2-1 or EN60998-2-2 E.g: Cembre Z205-1 Zetamini series Check the plate data before installation.
	Notes for installation The operation or safety of the light does not depend on the polarity or the power supply. For which, inverting the power supply cables does not pose a risk of malfunctioning.
	Notes for installation Support/lift the light using the external profile. Do not force the diffuser panel (white).

3.1 OVERALL DIMENSIONS

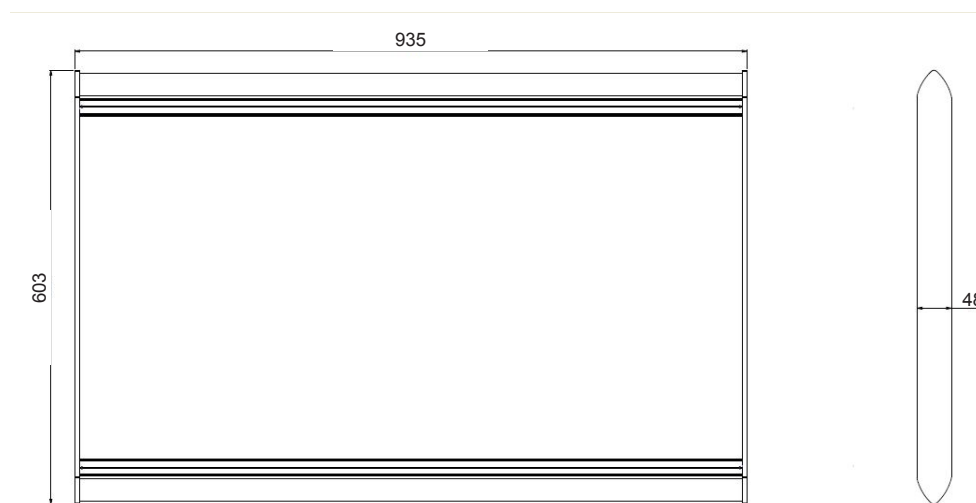
Fig. 2

3.2 POSITIONING

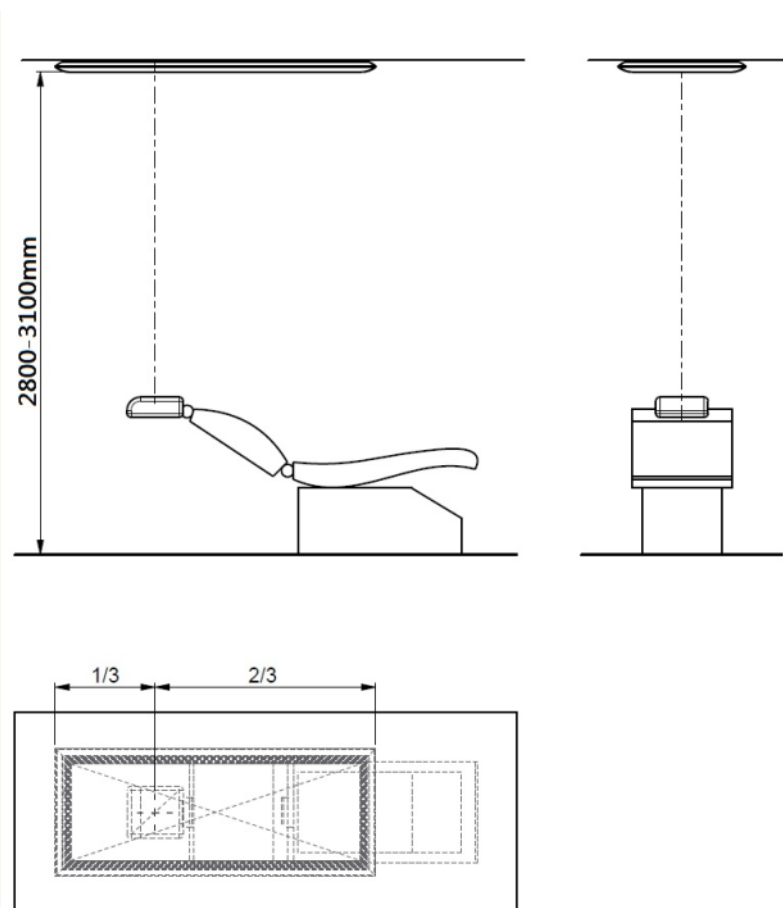
Fig. 3

3.2.1 SUSPENDED INSTALLATION

Using an accessory kit (not supplied), it is possible to install the light suspended from the ceiling. The positioning with respect to the chair remains unchanged (see point 3.2 fig.3)
For all installation information, refer to the instructions in the accessory kit.

3.3 CEILING INSTALLATION

The ZIVA lamp is supplied complete with ceiling installation elements.

INCLUDES:

- Lamp
- Safety sheet
- 2 Ziva lamp fixing brackets
- 2 Safety cables
- 6 fischer with Ø 6 mm screw

3.4 INSTALLATION PROCEDURE

1. Determine the size of the positioning axis of the magnets on the Ziva lamp.
2.  Drill the ceiling with a Ø6mm drill bit
Insert the plugs.
Fix the 2 fixing brackets to the ceiling, keeping the position dimensions of the magnets detected before, in order to allow the magnets the complete support.
3.  Secure the safety cables.
4. Connect the power cable.
5. Apply the lamp to the brackets (magnetic contact)
 Make sure the magnets are gripped on the brackets.
6. The ZIVA lamp is fixed to the ceiling.

	Note for installation Make sure the rawlplugs are securely in the ceiling by applying an axial load of at least 15 kg.
	Failure to comply with the requirements may result in the detachment of the product with risk of falling on persons.

4. INSTRUCTIONS OF USE

4.1 HOW TO USE

The lamp can be switched on and off using an omnipolar switch already present in the room and alternatively it can be switched on and adjusted using the “CASAMBI” application.

4.2 HOW TO USE “CASAMBI” APPLICATION

The “CASAMBI” application allows you to switch on, switch off, adjust the lighting and use configured settings.

- **Downloading the App**

The “CASAMBI” application (free-of-charge) can be downloaded from both the Play Store for Android devices and Apple for Apple devices.

The identification icon is:

Casambi

Casambi Technologies Oy

 PEGI 3


1. Connecting to the lamp


Once you have downloaded the App on your device, follow the instructions to install it. Once the installation is completed proceed with the opening of the application and switch on the lamp waiting a few seconds until the initial screen appears as in fig.4.

Fig. 4

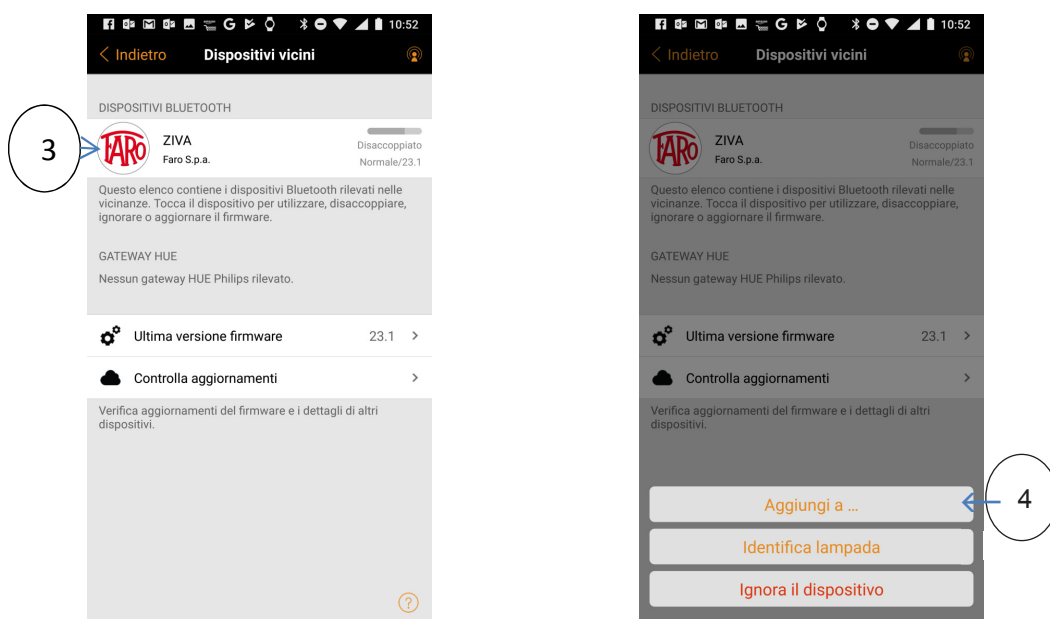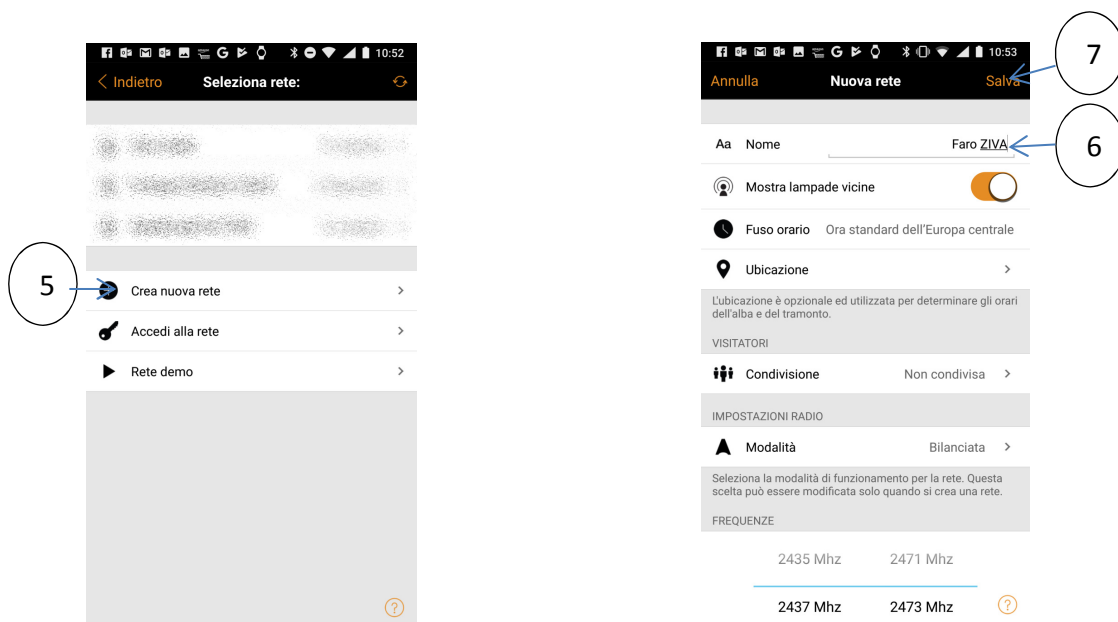
Once the bluetooth of the lamp and smartphone have been identified and paired, press on “Nearby devices” (fig.5) then follow the instructions from point 3 to point 7.

At this point the configuration is completed.

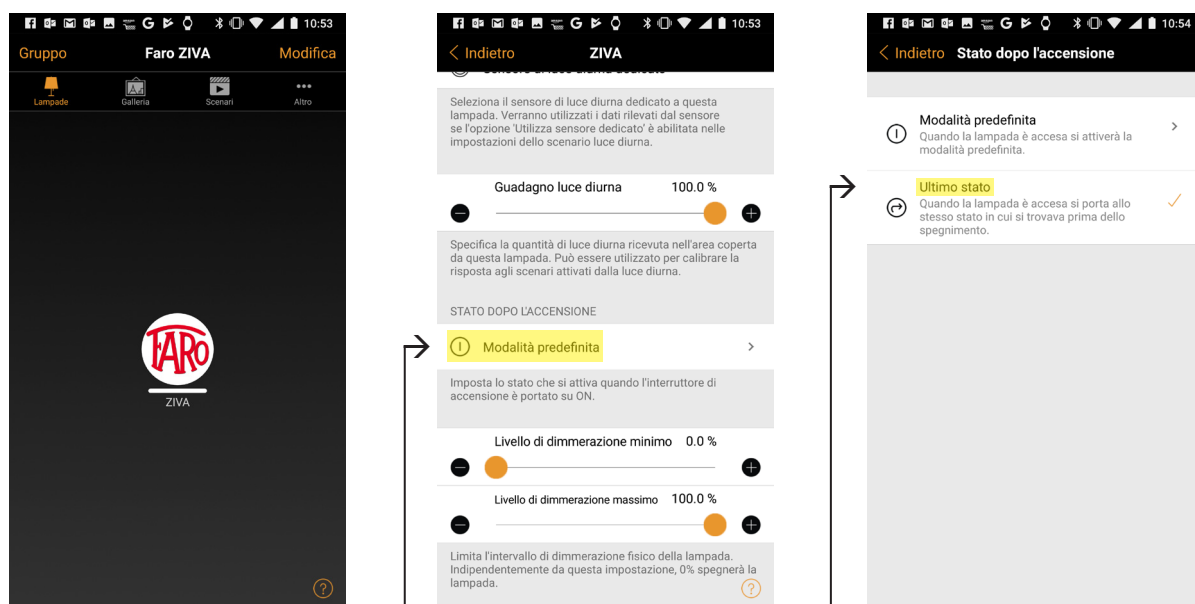HOW TO USE

Press twice on FARO to open the controls or press and hold to dim temporarily

Change the status after switching on to store the lighting settings

Select last status

5. MAINTENANCE

5.1 ORDINARY MAINTENANCE

There are no routine maintenance operations.

6. CLEANING

 	Warning against the risk of degradation and corrosion For all parts of the light made of metal or plastic it is strictly forbidden to use abrasive, acidic substances, containing chlorine or chlorine ions, detergents based on trichloroethylene, petrol, turpentine or similar. It is forbidden to spray any chemical substance directly on the device.
 	Warning When cleaning the parts, do not use detergents-disinfectants containing: • AMMONIUM HYDROXIDE • SODIUM HYDROXIDE • METHYLENE CLORIDE • METHYL ALCOHOL • ACIDS OF ANY KIND
	Note for cleaning Do not apply pressure to the panel during cleaning operations.

7. TROUBLESHOOTING

Effect	Cause	Action	Resp
The light does not light.	Lack of power supply.	Check the connection with the omnipolar circuit breaker.	User
		Check tha power supply is enabled.	User
		Check that the power supply is correct.	User
The light adjustments do not work.	The “CASAMBI” application does not work.	Reinstall the application.	User
	The distance between the smartphone and the Ziva is too far.	Move the smartphone closer to the light.	User
	The “CASAMBI” application is not connected to the Ziva light.	Retry connecting the light to the application.	User
	The mobile’s bluetooth is turned off.	Turn on the mobile’s bluetooth.	User
The light emitted by the Ziva light does not correspond to the options on the application.	The application settings have been changed.	Reset the options with the application as described in the instructions section.	User

For all other problems, contact Faro Spa authorized technical support.

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GUARANTEE CERTIFICATE

FARO offers the final customer a **24 month** guarantee starting from the date of purchase.

Repairs under guarantee must be performed at FARO; expenses and transport risks are at the risk of the purchaser. **Repair under guarantee is considered valid only when:**

- **all sections of the certificate have been filled in and sent in advance to FARO by Fax (039.6010540).**

The guarantee covers faults due to the bad quality of the material or manufacturing defects; in the case of valid claims, the guarantee covers free repair or replacement. **Claims for damages and/or interest are excluded.** The guarantee is not considered valid, at the sole discretion of FARO, if the fault is due to tampering, damage, incorrect use, improper maintenance and normal wear and tear.

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FARO S.p.A.
FARO SpA si riserva il diritto di modificare, senza preavviso, le caratteristiche indicate nel presente manuale.
FARO SpA reserves the right to change the specifications of this equipment without notice.
FARO SpA se reserve le droit de modifier, sans préavis, les caractéristiques dans ce manuel.
FARO SpA behält sich recht vor, jederzeit stillschweigend technische oder bauliche Änderung vorzunehmen.
FARO SpA se reserva el derecho de modificar sin aviso previo la características incluidas en el presente manual de uso.